medi GmbH & Co. KG
Medicusstraße 1
D-95448 Bayreuth
Germany
T +49 921 912-0
F +49 921 912-57
medi@medi.de
www.medi.de

medi Australia Pty Ltd
83 Fennell Street
North Parramatta NSW 2151
Australia
T +61-2 9890 8696
F +61-2 9890 8439
sales@mediaustralia.com.au
www.mediaustralia.com.au

medi Austria GmbH
Adamgasse 16/7
6020 Innsbruck
Austria
T +43 512 57 95 15
F +43 512 57 95 15 45
vertrieb@medi-austria.at
www.medi-austria.at

medi Belgium bvba
Posthoornstraat 13/1
3582 Koersel
Belgium
T +32 011 24 25 60
F +32 011 24 25 64
info@medibelgium.be
www.medibelgium.be

medi Brasil
Rua Itapeva 538 cj. 42
Bela Vista, São Paulo - SP
CEP: 01332-000
Brasil
Tel: +55 (11) 3500 8000
sac@medibrasil.com
www.medibrasil.com

medi Danmark ApS
Vejlegardsvej 59
2665 Vallensbæk Strand
Denmark
T +45 46 55 75 69
F +45 70 25 56 20
kundeservice@sw.dk
www.medit danmark.dk

medi Bayreuth España SL
C/Canigó 2 – 6 bajos
Hospitalet de Llobregat
08901 Barcelona
Spain
T +34 93 260 04 00
F +34 93 260 23 14
medi@mediespana.com
www.mediespana.com

medi France
Z.I. Charles de Gaulle
25, rue Henri Farman
93297 Tremblay en France Cedex
France
T +33 1 48 61 76 10
F +33 1 49 63 33 05
infos@medi-france.com
www.medi-france.com

medi Hungary Kft.
Bokor u. 21.
1037 Budapest
Hungary
T +36 1371 0090
F +36 1371 0091
info@medi.hu
www.medi.hu

medi Italia S.r.l.
Via Guido Rossa 28
40033 Casalecchio di Reno-Bologna
Italy
T +39 51 613 24 84
F +39 51 613 29 56
info@medi-italia.it
www.medi-italia.it

medi Medical Support Sdn Bhd
medi representative office Asia
Unit No. B-2-19, Block B, No.2,
Jalan PJU 1A/7A
Oasis Ara Damansara, PJU 1A,
47301 PETALING JAYA
Selangor Darul Ehsan
Malaysia
T +6 03 7832 3591
F +6 03 78323921
info@medi-asia.com
www.medi-asia.com

medi Middle East
P.O. Box: 109307
Abu Dhabi
United Arab Emirates
T +971 2 6429201
F +971 2 6429070
info@mediuae.ae
www.mediuae.ae

medi

medi Butler Off

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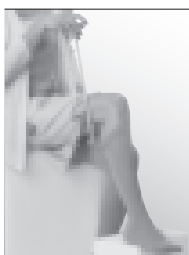
medi Butler Off

The medi Butler Off is extremely comfortable to use. The movement is simple to perform and does not require much strength. You yourself decide how quickly the movement is performed. With the doffing aid you can also achieve your goal slowly! The doffing aid can be used by both left- and right-handed users.

1. Slide the tip of the horn under the topband of the stocking.

2. Now push the handle gently downwards: the far end of the horn slides down along your calf and then over your heel. During the whole process the inner surface of the horn should remain in contact with your skin. At the same time the „tooth“ helps push the stocking downwards. In order to regain contact with your skin you can start again with the horn a little higher and then continue with the downward movement if necessary.

3. As soon as you reach your heel lift your heel up. Then tilt the stick downwards a little to guide the horn along the underside of your foot.

4. Now you can pick the stocking off the aid without having to bend down.

Caution

Make sure you do not press the stocking against the ground with the doffing aid. Otherwise the stocking may be damaged.

Now push the handle forwards gently: the stocking slides off your foot but stays on the horn.

Not recommended for mediven forte or any mediven flat knit models.

Benefits

- **independence**

If you have back problems or difficulties bending down then medi Butler Off will be a great help, no matter what age you are, and you can at last take your stockings off without assistance from others.

- **simple**

Its simple and safe handling will not cause you any problems, even if you have venous disorders (varicose veins, oedema) or painful legs.

- **gentle**

The doffing aid was developed and tested in co-operation with ergotherapists and doctors, is made of high-quality materials, and is manufactured particularly carefully. No pinching or scratching. When used properly it will not damage your compression stockings or other hosiery.

- **hygienic**

The doffing aid is hygienic. It is no problem to clean it with soap and water whenever necessary. Since the aid is robust it does not suffer wear-and-tear and will serve you well for years to come.

- **practical**

Thanks to its well-designed ergonomic features it is SIMPLE and PROBLEM-FREE to use and fits in any bag.

Further indications for use:

The medi Butler Off is also eminently suitable for helping you take off normal socks and stockings or as a shoehorn.

Use as a Shoehorn

Slide the far end of horn between the shoe and the underside of your foot (not more than 1 – 2 cm).
Now push your foot downwards: thanks to the ergonomic shape of the horn your foot glides easily into the shoe!

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